## EN Whitening with 10% carbamide peroxide with a reduced daily use time in patient with dental sensitivity

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### EN CLINICAL/RADIOGRAPHIC EXAM

During the anamnesis, it was reported that she had never performed a whitening treatment before, and had a generalized mild degree of dental sensitivity. The physical examination revealed the lack of previous restorations, gingival health, absence of carious lesions and an initial A3 shade, according to the VITA Classical shade guide.

### TREATMENT

The proposed treatment was supervised at-home whitening with 10% carbamide peroxide gel (Whitness Perfect 10%) with a reduced daily use time, for 1 hour per day, during 21 days. Treatment was started after the preparation of silicone trays and guidance about the use of the whitening gel, diet and hygiene. The patient was advised that in case of exaggerated sensitivity, she should get in touch for a reassessment of the proposed treatment. During the control of half of the treatment, the A1 shade was taken, showing significant whitening without complaint of sensitivity, allowing the continuity of the treatment. After 21 days, the patient ▶

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appeared to be satisfied with the visual gain achieved (final shade B1), as she did not have to interrupt the treatment due to the sensitivity.

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STEP BY STEP | PASO A PASO

1. Initial intraoral appearance.  
1. Aspecto intraoral inicial.

2. Initial shade taking equivalent to the A2 shade according to the VITA guide.  
2. Tomada de color inicial equivalente a un color A2 siguiendo la guía VITA.

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// The whitening treatment with a reduced time of daily use promoted a significant shade change at the end of 21 days of continuous treatment, with a satisfied patient and without dental sensitivity that would make treatment unfeasible. //

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Prof. Dr. Fabiano Carlos Marson

FGM MATERIALS USED | MATERIALES FGM UTILIZADOS

TAKE-HOME WHITENING GEL  
(CARBAMIDE PEROXIDE)  
Whiteness Perfect 16%

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PLATES FOR TRAYS  
Whiteness Plates for  
Custom Dental Trays

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