EN Teeth whitening with the combined technique: 3-year follow-up

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26-year-old female patient.

COMPLAINT

Yellowish teeth with a lot of difference between anteriors and posteriors.

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EN CLINICAL/RADIOGRAPHIC EXAM

Good hygiene, healthy teeth, lack of sensitivity. Presence of incipient white lesions that could lead to unwanted attention.

TREATMENT

A combined technique of take-home whitening with 10% carbamide peroxide (Whiteness Perfect) and 35% hydrogen peroxide (Whiteness HP Blue) was used. Treatment carried out in 21 days, following the protocol described below:

Session 01: Prophylaxis, molding and tray installation. Initial whitening with Whiteness Perfect 10%.

Session 02 (seventh day): Evaluation, questioning about sensitivity, which resulted negative. First application of Whiteness HP Blue 35% for 30 minutes with no gel change. ▶

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▣ **Session 03 (fourteenth day):** Evaluation, questioning about sensitivity, which resulted negative. Second application of Whiteness HP Blue 35% for 30 minutes with no gel change.

Session 04 (twenty-first day): Final evaluation, polishing with Diamond felt disks and Diamond Excel polishing paste. Application of Flúor Care for 5 minutes.

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STEP BY STEP | PASO A PASO

1 and 2. Initial aspect of the case presenting two challenges: balancing the tone between anteriors and posteriors as well as maintain the opacity of white lesions unaltered.

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3 and 4. The process of forming and construction of the tray must undergo a molding process in good quality alginate and application of vacuum at the right moment of silicone plastification. Early or delayed movements can lead to poor tray adaptation, decreasing the quality of results. The patient started with the take-home whitening for 7 days.

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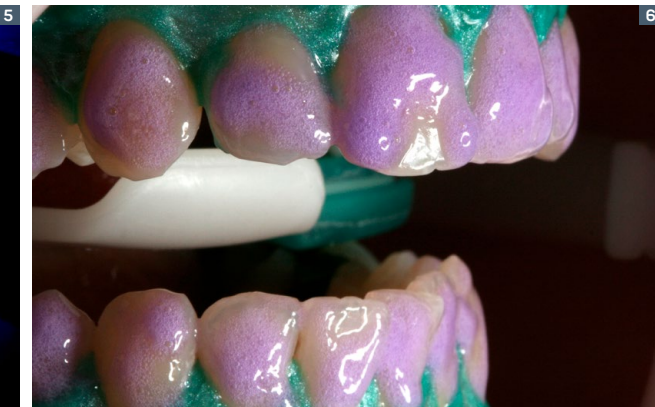
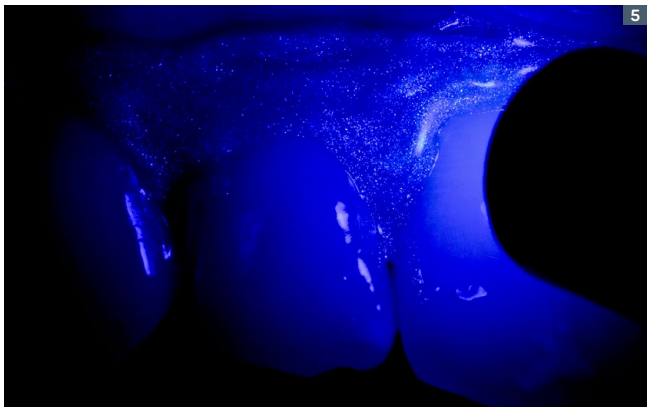
// Combined techniques supervised by the dentist promote interesting results and establish a relationship of trust and care with the patient. The combination of Whiteness HP Blue 35% and Whiteness Perfect 10% has been one of our successful protocols in recent years. Establishing follow-up appointments add value to the procedure and allows us to efficiently control treatment. //

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Dr. Marcelo Alves

5. First session with Whiteness HP Blue: application and light-curing of the gingival barrier. It is an important step, because gel leaks in contact with the gum may interrupt the treatment, increasing working time.

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6. Note the excellent thixotropy of the gel and its ease of application. Control and stability allow a selective application to be made, preventing contact of the gel with the white lesion.

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7. Final appearance after the first session with Whiteness HP Blue and 7 days with Whiteness Perfect 10%. White stains, when in contact with peroxide, dehydrate and become opaque thus generating a stained appearance. Controlled side effect.

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8 and 9. Final appearance of the case after 21 days.

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FGM MATERIALS USED | MATERIALES FGM UTILIZADOS

IN-OFFICE WHITENING GEL
 (HYDROGEN PEROXIDE)
Whiteness HP Blue

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PLATES FOR TRAYS
Whiteness Plates for Custom Dental Trays

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TAKE-HOME WHITENING GEL
 (CARBAMIDE PEROXIDE)
Whiteness Perfect 10%

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LIGHT-CURING GINGIVAL BARRIER
Top dam

BARRERA GINGIVAL FOTOCURABLE
Top dam

FELT DISKS
Diamond

DISCOS DE FIELTRO
Diamond

SODIUM FLUORIDE
 AT 1.23% FOR TOPICAL
Flúor Care

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whiteness HP Blue

35% hydrogen peroxide-based whitening gel for in-office use.

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Single application of 45 minutes per session: proved pH control and stability during the whole gel application.

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Remineralizing formula: the presence of calcium in the formula prevents demineralization, reducing sensitivity.

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Easy mix: easy to mix the phases (peroxide + thickener) with the application directly from the syringe.

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