EN A simple way to solve complex cases: the semi-direct approach

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Dolor intenso en el diente 27.

EN CLINICAL/RADIOGRAPHIC EXAM

During the clinical and radiographic examination, a large carious lesion was found in the distal portion of the tooth, already invading the pulp chamber.

TREATMENT

Endodontics was performed on tooth 27 and then restorative planning was done. For this tooth with such large cavity, the preparation of the palatine conduit was used to place a fiberglass post (Whitepost DC 0.5). In the conduit, Ambar Universal APS adhesive was applied by the self-etching technique and then Allcem Core cement was applied for post cementation. After that, the entire cavity was filled with the same cement. In the same session, the tooth received an onlay and molding preparation. After obtaining the plaster model, an onlay restoration was made with Vittra APS composite. After internal treatment with Condac Porcelana 5% for 30 seconds, Prosil and Ambar Universal, the onlay was cemented with Allcem Core cement, color A1.

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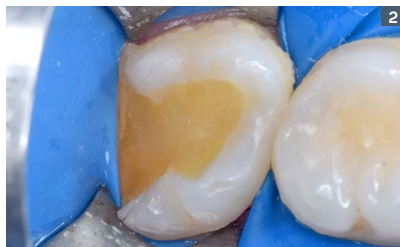
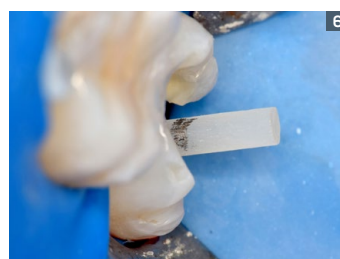
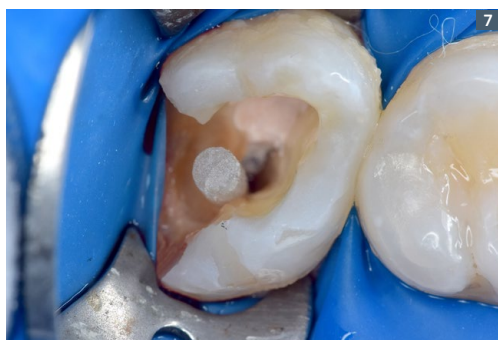
8. Application of Ambar Universal APS adhesive inside the conduit, followed by light-curing.
9. Insertion of Allcem Core cement with the aid of a microapplicator.

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// Restoring heavily destroyed teeth is always a great challenge, mainly in the mechanical aspect. FGM's product line, with all its variety, precision and quality, allows the resolution of this type of case with extreme safety. //

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Prof. Dr. Fabio Sene.

10. Whitepost cemented.
10. *Whitepost cementado.*

11. Along with post cementation, cavity filling with the same product (Allcem Core).

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15. Final result after cementation with Allcem Core cement in color A1.

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FIBERGLASS POST
Whitepost

POSTE DE FIBRA DE VIDRIO
Whitepost

DUAL-CURING RESIN CEMENT
Allcem Core

CEMENTO RESINOSO DE CURADO DOBLE
Allcem Core

LIGHT-CURING COMPOSITE
Vitra APS

RESINA FOTOCURABLE
Vitra APS

7TH GENERATION COLORLESS
LIGHT-CURING ADHESIVE
Ambar Universal APS

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HYDROFLUORIC ACID AT 5%
Condac Porcelana 5%

ÁCIDO HIDROFLUORÍDRICO AL 5%
Condac Porcelana 5%