## EN Achieving exceptional results with direct restorations

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58-year-old female patient.

### COMPLAINT

Dissatisfaction with teeth aesthetics in relation to wear, defective restorations and the presence of diastema.

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### EN CLINICAL/RADIOGRAPHIC EXAM

Generalized presence of wear on anterior teeth, defective restorations, premolar abfractions and the presence of diastema.

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### TREATMENT

Study models and laboratory waxing were carried out to make a silicone guide. The proposed treatment was the making of direct veneers in composite (Vittra APS) as it was a quick, non-invasive and repairable treatment. The choice of a good quality composite is of utmost importance, and we can achieve aesthetic results similar to ceramics, with high mechanical strength and shine longevity.

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STEP BY STEP | PASO A PASO

1. Initial smile.  
1. Sonrisa inicial.

2. Initial intraoral appearance.  
2. Aspecto intraoral inicial.

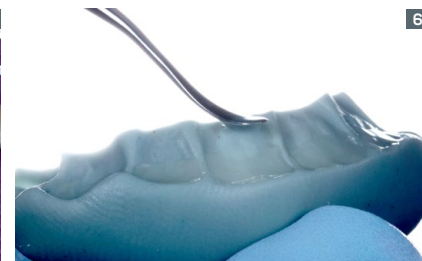
3. Intraoral aspect of the upper arch.  
3. Aspecto intraoral de la arcada superior.

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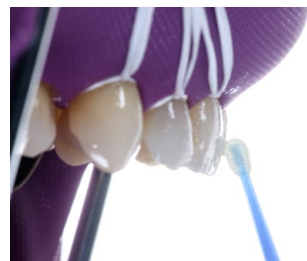
13. Result after positioning the composites.

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14. Re-fitting the isolation on the premolars in order to allow a better adaptation in the cervical region.

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15. Application of a Opallis layer in shade Opaque Pearl, in order to mask the darkened dentin in the cervical region.

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// Vittra APS surprised me with its incredible monomer and light-curing technology, and surprised me even more by its ease of handling. I believe that the success of a composite is closely linked to the union of composition factors with their rheological characteristics. Its polishing is sensational. //

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17. Restorations finished in the upper arch before finishing and polishing.

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18. Finishing performed with Diamond Pro sandpaper disks.  
19. Final polishing with felt disks (Diamond Flex) and Diamond Excel polishing paste.

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**FGM MATERIALS USED | MATERIALES FGM UTILIZADOS**

LIGHT-CURING COMPOSITE  
**Vittra APS and Opallis**

RESINA FOTOCURABLE  
**Vittra APS y Opallis**

5<sup>TH</sup> GENERATION  
 LIGHT-CURING ADHESIVE  
**Ambar APS**

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