EN Using a 3-in-1 cement for prosthetic rehabilitation

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41-year old male patient.

COMPLAINT

Tooth 36 with a fractured restoration.

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EN CLINICAL/RADIOGRAPHIC EXAM

Tooth 36 showed satisfactory endodontic treatment, without painful symptomatology. The existing restoration showed bordering fractures, recidivist caries and invasion of the biological space.

TREATMENT

A surgery to recover the biological distances was performed. A fiberglass post was installed and a core was built up. Then, a ceramic overlay was cemented. All the steps were performed with Allcem Core, a multifunctional dual-curing resin cement.

// *Allcem Core is an excellent restorative option because it is an extremely versatile and reliable material. FGM always innovates in its line of products and, in this case, made available a material that exemplifies perfectly the philosophy of the company. Allcem Core is part of my daily work at the dental office and also of my classes.* //

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
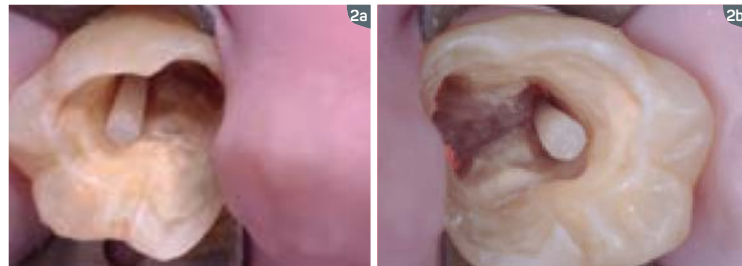
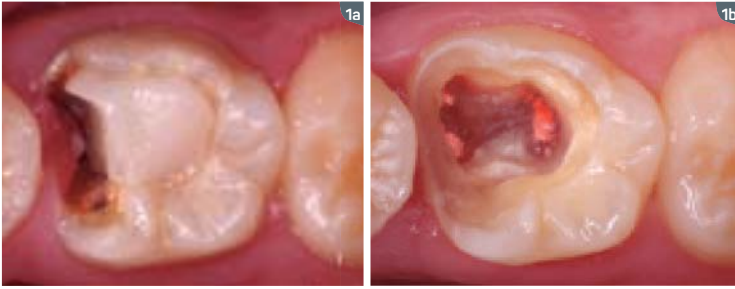
2a and 2b. Absolute isolation and test of the fiberglass post Whitepost DC1, after conduct preparation with Whitepost drills.

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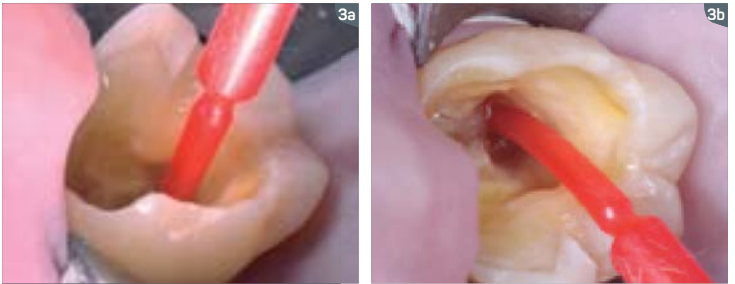3a to 3c. After total etching, application of Ambar adhesive with Cavibrush and light curing for 40 seconds.

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4a and 4b. Application of the Allcem Core cement in the interior of the conduct (A3 shade).

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

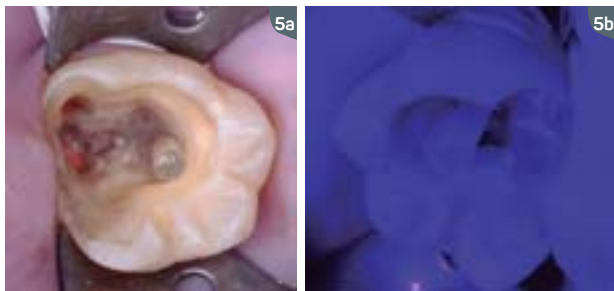
5a and b. Post inserted in the conduct in the cement and light cured with high-power LED for 2 minutes.

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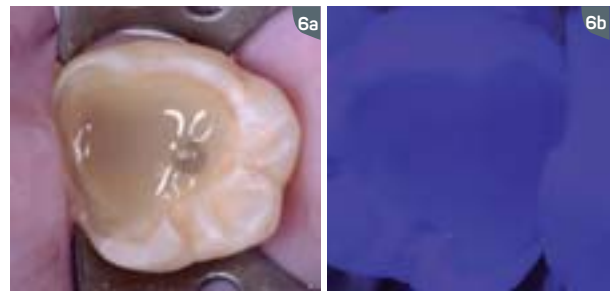6a and b. Core built up with Allcem Core and light cured for 40 seconds.

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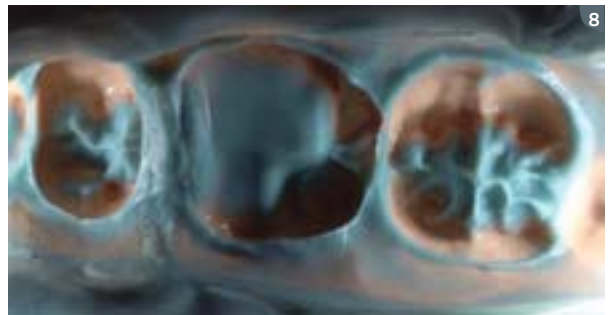7. Preparation for overlay and insertion of gingival retraction cord for molding.

8. Molding with addition silicone.

9. Building temporary teeth.

10. Model with trimmed dies and prosthesis in feldspathic ceramics finished (DPT André Ferraz - Araraquara/SP, Brazil).

11. Prosthetic piece placed on tooth, showing no need for adjustments.

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12a. Etching of the piece with Condac Porcelana 10% for 2 minutes.

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12b. Etching of the piece with Condac 37 for 1 minute.

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12c. Application of silane Prosil and wait time of 5 minutes.

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13a. Absolute isolation and roughing of the core.

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13b and 13c. Acid etching with Condac 37 for 30 seconds, application of Ambar adhesive and light curing.

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14. Application of Allcem Core in the interior of the piece.

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15. Positioning of the ceramic piece on the tooth followed by compression.

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FIBERGLASS POST
Whitepost

POSTE DE FIBRA DE VIDRIO
Whitepost

DUAL-CURING RESIN CEMENT
Allcem Core

CEMENTO RESINOSO DE CURADO DOBLE
Allcem Core

5th GENERATION LIGHT-CURING
ADHESIVE
Ambar

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HYDROFLUORIC ACID AT 10%
Condac Porcelana 10%

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