ViroSurf^{zero} wipes

Surface Disinfectant Alcohol-free

For the surface disinfection of non-invasive medical devices

INSTRUCTIONS FOR USE

User Tub

Remove cap of dispenser tub

Cut open air tight plastic pouch across the top of the pouch.


Refill Pouch


Refill pouches can be used with or without a tub. For pouches without a ziplock ensure pouch is sealed between use to prevent evaporation.


Cut open air tight plastic pouch across the top of the pouch.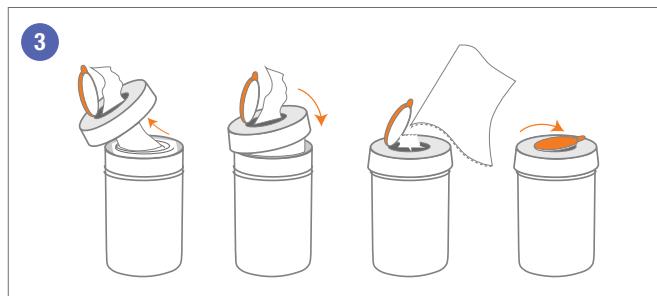


Feed the centre wipe through the hole in the cap and replace. Pull wipe at a 45° angle.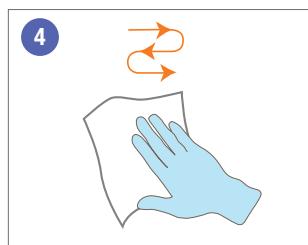


Using a new wipe for each application
wipe objects to be disinfected so that they are completely moistened.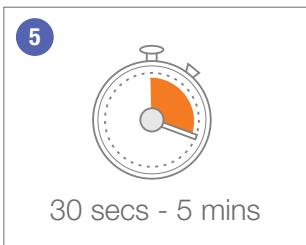


Let the disinfectant take action over a minimum of 30 seconds (to achieve full efficacy leave for 5 minutes), allowing surface moisture to evaporate.
Do not use after expiry date.
For professional use only.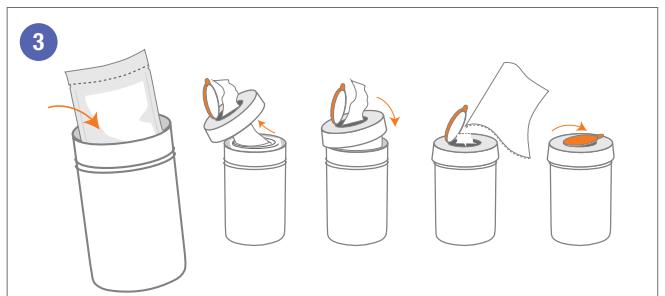

Feed the centre wipe through the hole in the cap and replace. Pull wipe at a 45° angle.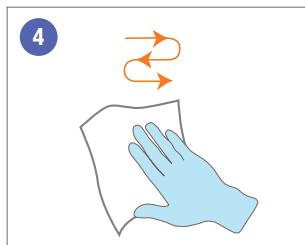

Using a new wipe for each application
wipe objects to be disinfected so that they are completely moistened.

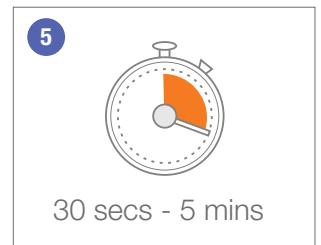Let the disinfectant take action over a minimum of 30 seconds (to achieve full efficacy leave for 5 minutes), allowing surface moisture to evaporate.
Do not use after expiry date.
For professional use only.

Surface Disinfectant Alcohol-free

Composition

Aqua, Benzyl-C12-18-Alkyldimethyl Chlorides, C12-14-Alkyl[(Ethylphenyl)Methyl] Dimethyl Chlorides, Didecyldimethylammonium Chloride, 2-Propanol (not an active, <1%), Surfactant, Dipropylene glycol monomethyl ether, Polyhexamethylene biguanide hydrochloride, Cocamidopropyl Betaine, Tetrasodium EDTA, Antifoam, Preservative, Aroma.

Use on a wide range of surfaces

Glass - safe on all types of glass
Metal - including aluminium, stainless steel and coated steel
Plastic - including PE, PP, nylon and acrylics
Corian - including other synthetic surfaces

Tested to Standards

EN 14885, EN 14476, EN 14348, EN 13727, EN 13704, EN 13624.

Order Codes

VIR0030 - Kit = 1 x dispenser tub & 12 x refills
VIR0055 - Kit = 1 x dispenser tub & 6 x refills
VIR0052 - Refill Pack
VIR0039 - Empty dispenser tub

Each refill pack contains 250 'Extra Value' wipes made from a 21gsm high quality material, each wipe measures 200 x 145mm.

UK WARNING: P101: If medical advice is needed, have product container or label at hand, P102: Keep out of reach of children. P301+P312: IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. The wearing of protective gloves is recommended. Storage: P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. Contraindications. Incompatible materials: Aluminium. Copper. Disposal procedures. Do not discharge liquid into drains or rivers. Do not flush. Non maceratable.

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