
# Quantum

## Instrument Disinfectant


### Colour Change Powder

Enzymatic powder based disinfectant effective against viruses including: HBV, HCV, HIV, Adenovirus, Norovirus and Coronavirus


### INSTRUCTIONS FOR USE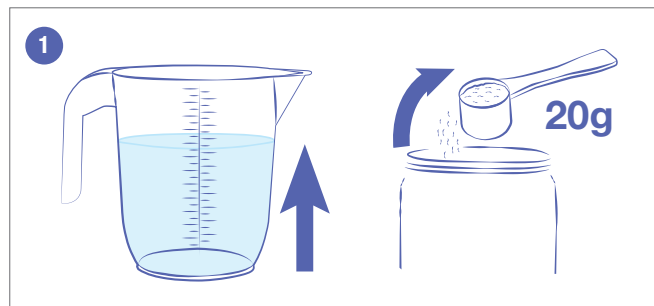


Measure 1 litre of warm tap water (30-40°C) and 1 level scoop of powder (20g ± 2g)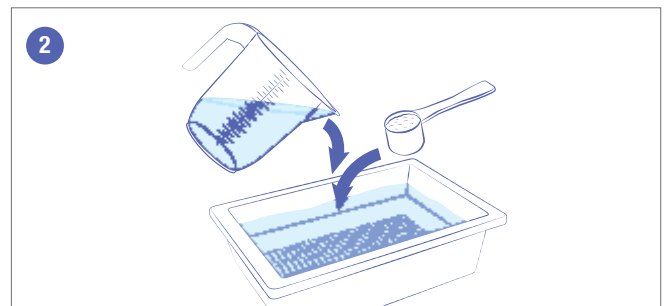


Place both water and powder into the instrument bath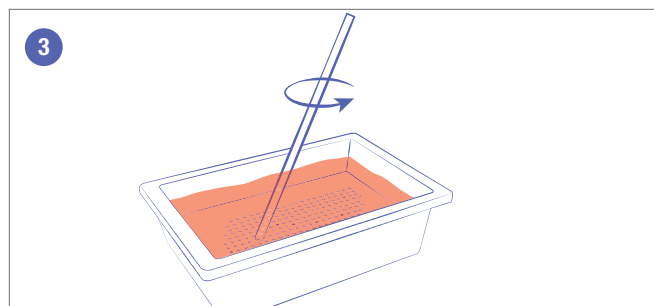


Mix for 60 seconds and the solution will turn red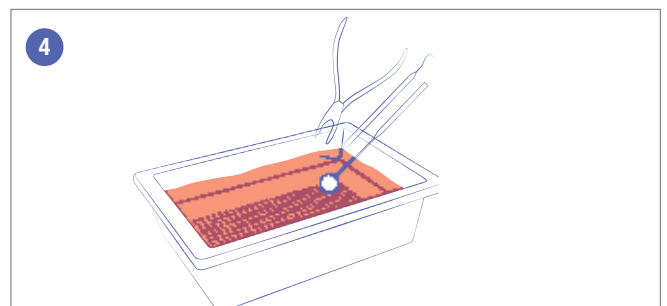

Completely immerse the instruments into the solution ensuring that no air is left within hollow instruments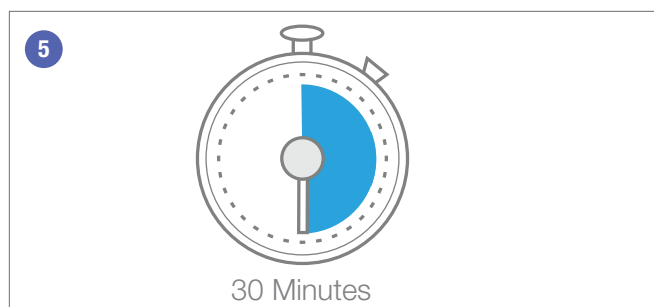

Leave for 30 minutes. Do not exceed this recommended time. Change solution daily. For professional use only. See product labels and MSDS for further safety guidelines and storage conditions. Available at [topdental.co.uk](http://topdental.co.uk)

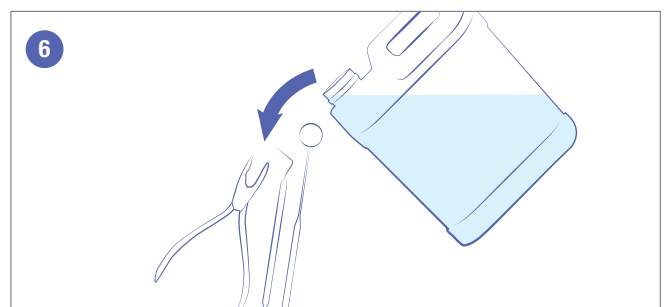After disinfection, rinse instruments thoroughly with RO or distilled water prior to further processing and sterilisation. The solution will remain effective for 8 hours. Change to fresh solution daily or earlier should physical dirt in the solution become evident. We recommend preparing fresh solution at the start of the working day. When the solution is colourless it is no longer active.

# Quantum

## Instrument Disinfectant

### Colour Change Powder

### Composition

TAED, Sodium Percarbonate, Citric Acid Monohydrate, Sodium Alkyl Aryl Sulphonate, Sodium Sulphate Anhydrous, Tetrasodium (1-hydroxyethylidene) bisphosphonate, Subtilisin, Sodium Nitrite, Dye.

### Tested to Standards

EN 14885, EN 14476, EN 13727, EN 13704, EN 13697, EN 13624

### Order Code

VIRQ700 - 700g container with scoop x 9 per box

### UK Warning

**DANGER** H302: Harmful if swallowed. H318: Causes serious eye damage.

P280: Wear protective gloves/protective clothing/eye protection/face protection. P264: Wash hands thoroughly after handling. P301+312:

IF SWALLOWED: Call a POISON CENTER/doctor if you feel unwell.

P305+351+338: IF IN EYES: Rinse cautiously with water for several

minutes. Remove contact lenses, if present and easy to do. Continue

rinsing. P233: Keep container tightly closed. P232: Protect from moisture.

P403+P235: Store in a well-ventilated place. Keep cool.

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